



ALTRA-GEL PADS

USE, CARE, and CLEANING INSTRUCTIONS

About

Bryton's Altra-Gel's formula is a specific 100% dry viscoelastic polymer that has been specially developed to give ultimate anti-shear protection. This product is intended for medical use only, indoors, in dry and sterile conditions.

Altra-Mats - Latex free, reusable, fluid proof, and easy to clean.

READ USE, CARE, and CLEANING INSTRUCTIONS BEFORE USE!

Distributed by: Bryton Corporation



4001 Methanol LN
Indianapolis IN, 46268
www.brytoncorp.com
info@brytoncorp.com

Reordering Information

For reordering Altra-Gel pads please call 1-800-567-9500, visit www.brytoncorp.com, or email sales@brytoncorp.com.

General Use of Gel Pads

FAILURE TO FOLLOW CARE INSTRUCTIONS WILL VOID YOUR WARRANTY.

- In line with good clinical practice, assess the patient's condition and areas of potential risk prior to deciding which gel pads and positioners to use.
- Avoid placing sharp instruments on gel pads as they may puncture the film.
- Gel pads may be heated to body temperature in a hospital grade blanket warming chamber, or on a heated mattress. Do not heat above 40°C. Please follow instructions of warming equipment or mattress manufacturer as appropriate.
- Avoid pulling table pads while patients are on them, as excessive force may adversely affect the product.
- Avoid pulling gel pads along rough or sharp surfaces.
- Do not cut or tear gel pads, keep sharp objects and knife blades away from gel skin.
- Avoid exposing the products to naked flame or excessive temperature.
- Use correct lifting techniques when lifting heavy products.
- Freezing of gel pads with Velcro attachments is not advisable as this may affect the adhesion of the Velcro to the product.
- Do not apply tape or adhesive products to the skin, as it will leave residue.

Storage of Gel Pads

- Where possible, store gel pads flat.
- Fold large table pads in sections of approximately 12".
- Avoid placing heavy loads or sharp objects on gel pads.
- Maintain temperature of gel products below 45°C and above -18°C.
- Lay flat in warm area to reduce fold/wrinkle marks, some wrinkles on skin are normal.

Cleaning of Gel Pads

- Gel pads can be easily cleaned using standard operating room cleaning agents.
- Avoid soaking products for long periods.
- Do not use corrosive chemicals or strong solutions on the gel skin.
- Keep Velcro and adhesive patches clean, as they may attract dust, hair and foreign matter.

Warranty of Gel Pads

- Gel pads are manufacturer warranted against defects in materials for 1 year.
- Alterations, knife cuts, and abuse void manufacturer warranty.
- Warranty does not include damage caused from excessive stretching, twisting or heat.
- Use of gel positioners for other than intended purpose voids warranty.

