



E-Z-Tranzfer Boards & Devices
Operation - Maintenance

⚠ CAUTION

Read and understand all of the instructions and safety information in this manual before operating this product.



Table of Contents

Summary	1
Models/Specifications	2
Use/Placement	3
Cleaning, Care and Disinfection	3
3.1 Blue bottom cover	3
3.2 Silver cover.....	3
3.3 Board	3
Warranty	4
Visual and Function Inspection/Safety Information	4
Product Labeling.....	4
Disposable Information	4
Accessories	4

Summary

Description

E-Z-Tranzfer Boards – the gold standard in patient transfer. These lightweight transfer boards are designed to transfer patients with ease, preventing back injury to the caregiver. The dual handle grip design is a must for patient safety and caregiver assistance, preventing injuries caused by off-balance tipping of one handle transfers. E-Z-Tranzfer boards have been designed for lateral or fowler transfer.

Purpose of this manual

This manual is to provide the user instructions in the operation and maintenance of the E-Z-Tranzfer devices by Bryton Corporation. This manual also contains general specifications, warnings and cautions.



Please read the instructions carefully before using for the first time!

Warning means: failure to follow these instructions may cause damage to the product or injury to the patient or other persons.

Models/Specifications

Specification	PT-6840	PT-6870
Dimensions:	37"L x 20"W Foldable: 19"L x 20"W	69"L x 20"W Foldable: 36"L x 20"W
Material		
Cover Material:	Polyethylene, latex free low-friction fabric w/coated silver cover	Polyethylene, latex free low-friction fabric w/coated silver cover
E-Z-Tranzfer Board Load:	Max: 440 lb	Max: 440 lb
Radiolucent:	X-Ray Compatible	X-Ray Compatible
Includes:	E-Z-Tranzfer Board, Cover, & Manual	E-Z-Tranzfer Board, Cover, & Manual



max. 440 lb

Cover:



Use/Placement

The Bryton E-Z-Transfer Board is ideally suited for traumatic, painless and low-friction transfer of seated and recumbent, e.g. heavy, particularly helpless or premedicated patients to a mobile shower stretcher, a different bed, a stretcher, different types of wheelchairs, an x-ray, examination or operation table. Transfer is also possible over a short distance or over different heights. The Bryton E-Z-Transfer Board is extremely flexible with its support for the transfer of patients with many drainages or with problems such as respiratory distress syndrome. For safety reasons, transfers with the Bryton E-Z-Transfer Board should be done with two persons and if applicable with a sheet for pulling the patient.

1. Move the patient in a recumbent position slightly to the side and slide the Bryton E-Z-Transfer Board about 1/5 of the way under the patient.
2. Place the patient in a supine position on the Bryton E-Z-Transfer Board.
3. Pull the patient over and onto the Bryton E-Z-Transfer Board in the transfer direction with the pull sheet.
4. Without the pull sheet stand beside the transfer level and push the patient onto the Bryton E-Z-Transfer Board in the transfer direction

Warning

- Ensure the working heights of the transfer levels are correct for working without back injury.

To remove the Bryton E-Z-Transfer Board place the patient briefly on the side and remove the Bryton E-Z-Transfer Board from under the patient. The Bryton E-Z-Transfer Board should be used by four persons when transferring bariatric patients. The Bryton E-Z-Transfer Board can also be briefly lifted with a load during a transfer to negotiate an obstacle. The Bryton E-Z-Transfer Board can be pulled up or down, moved or briefly lifted to move it away from an unfavorable transfer situation. Always be aware of your back when working as a carer.

Note: This product is suitable for re-use.

Option: The Bryton E-Z-Transfer Board can also be used with the disposable sliding foil. The disposable sliding foil is wound up on a roll with perforated foil pieces of 70"L. The disposable sliding foil pieces can be singularly and simply separated from the foil. The disposable sliding foil can be halved to 35"L with foil cutter and is used for the Bryton E-Z-Transfer Board PT-6840. Pull the disposable sliding foil over the Bryton E-Z-Transfer Board. With its outstanding sliding properties the disposable sliding foil used with the Bryton E-Z-Transfer Board makes the most difficult transfer easy.

Cleaning, Care and Disinfection

The Bryton E-Z-Transfer Board must be cleaned with medical disinfectants before and after every change of patient.

3.1. Blue bottom cover

Do not use abrasive or bleaching agents or sharp objects for cleaning. The use of bleach- or alcohol-based cleaning agents can permanently impair the functional properties of the blue cover. The blue cover can be easily disinfected using oxygen-based disinfectant solutions. For use in hygienically sensitive areas, please use the disposable sliding foil.

3.2. Silver cover

The use of bleaching agents can permanently impair the functional properties of the silver cover. The silver cover can be easily disinfected with alcohol-based disinfectant solutions.

3.3. Board

The board itself is very durable and can be cleaned with standard household cleaning products. The board may easily be disinfected with alcohol-based disinfectant as well. Allow the board to dry after disinfection.

It is not recommended to use the autoclave due to the thermo-sensitive structure of the material.

Note: When using the Bryton E-Z-Transfer Board with the disposable sliding foil (the sliding foil is disposable and not washable), please note the special instructions of the sliding foil.



Option: Use of the disposable sliding foil.

Warranty

The warranty is valid for a period of 2 years. It does not cover any defects which are due to improper use. Bryton Corp. can not be liable for any accidental or indirect injury or damage.

Visual and Function Inspection/ Safety Information

Check the equipment regularly and carefully to make sure it is working properly. Make sure that the products are in good condition and that the material and seams are not damaged.



Warning

- *Faulty products must no longer be used.*
- *Make sure that there are a sufficient number of carriers and assistants corresponding to the transfer situation.*
- *Never leave the patient unattended.*
- *The smooth surface considerably minimizes resistance to friction during the change of position. Carefully check the two transfer surfaces, making sure they are stable and brakes are applied (wheelchair, bed, etc.).*
- *The distance between free transfer surfaces using the Bryton E-Z-Tranzfer Board must not be more than 5.9"*
- *Particular when lifting the Bryton E-Z-Tranzfer Board make sure that the patient is securely on the Bryton E-Z-Tranzfer Board and cannot slide off.*
- *Make sure that carriers are always aware of their backs during the transfer.*
- *Note the weight limit (440 lb).*
- *Do not use sharp-edged items when using the Bryton E-Z-Tranzfer Board. The Bryton E-Z-Tranzfer Board could get damaged in this case.*
- *Transfers under uncovered skin must be done with a cloth between the Bryton E-Z-Tranzfer Board and the patient (e.g. towel, blanket)*

Product Labeling

The Bryton E-Z-Tranzfer is a medical product, class I and complies with the MDD directive 93/42/EEC.

Disposable Information

For the Bryton E-Z-Tranzfer there are no special disposable rules; it can be disposed of in accordance with national and local regulations as necessary.

Accessories

Disposable Sliding Foil

If you have any questions about the use of our products, please call our service number: 800-567-9500

Note: Information subject to technical modifications to our products and deviations in color or printing errors.

