



SURGICAL TABLE PADS

USE, CARE, and CLEANING INSTRUCTIONS

About

Bryton's surgical table pad surfaces are 100% latex free vinyl fabrics that have been specially developed to give anti-shear protection. This product is intended for medical use only, indoors, in dry and sterile conditions.

Bryton's Pads - Latex free, reusable, fluid proof, and easy to clean.

READ USE, CARE, and CLEANING INSTRUCTIONS BEFORE USE!

Manufactured by: Bryton Corporation



4001 Methanol LN
Indianapolis IN, 46268
www.brytoncorp.com
info@brytoncorp.com

Reordering Information

For reordering Bryton Surgical Table Pads please call 1-800-567-9500, visit www.brytoncorp.com, or email sales@brytoncorp.com.

General Use Guidelines for Bryton's Surgical Table Pads

FAILURE TO FOLLOW CARE INSTRUCTIONS WILL VOID YOUR WARRANTY.

- In line with good clinical practice, assess the patient's condition and areas of potential risk prior to deciding which Bryton Surgical Pad Set Series to use.
- Avoid placing sharp instruments on surgical pads as they may puncture the vinyl fabrics.
- Do not apply tape directly to surface of pads, as it will leave residue.
- Always inspect pads exterior covers and seams prior to use for any wear, tear, damage, or contamination. Do not use damaged or contaminated pads.
- Avoid pulling table pads while patients are on them, as excessive force may adversely affect the product.
- Avoid pulling pads along rough or sharp surfaces.
- Do not cut or tear pads, keep sharp objects and knife blades away from vinyl fabric surfaces.
- Avoid exposing the products to naked flame or excessive temperature.
- Freezing of gel pads with Velcro attachments is not advisable as this may affect the adhesion of the Velcro to the product.
- Avoid "pooling" of cleaning agents or rinses on the surface of the pads.
- Important: Carefully towel pads dry, do not allow to air-dry.

Storage of Pads

- Where possible, store Bryton's surgical table pads flat.
- Do not fold table pads or stack laterally.
- Avoid storage of heavy loads or sharp objects on pad sets.
- Maintain temperature of pads below 45°C and above -18°C.

Cleaning & Disinfecting of Pads

Cleaning (Soils & Stains)

- Wash with a generous application of neutral soap suds and lukewarm water. Rinse with water and dry with a clean cloth, do not air-dry. Do not immerse pads.

- Or

- Use a diluted bleach solution of 1-part bleach to 10-parts water. Wipe dry with a clean cloth, do not air-dry. Do not immerse pads. IMPORTANT: BLEACH MUST BE DILUTED TO THE 1:10 RATIO.
- Do not use corrosive chemicals or strong solutions on pads.
- Keep Velcro clean, as they may attract dust, hair & foreign matter.

Disinfecting

- Use of quaternary cleaners is permitted. Do not allow to pool or air-dry on surfaces of pads.
- Or
- Use a diluted bleach solution of 1-part bleach to 10-parts water. Wipe dry with a clean cloth, do not air-dry. Do not immerse pads. IMPORTANT: BLEACH MUST BE DILUTED TO THE 1:10 RATIO.

Note: All chemical/solutions must be properly diluted before use, not doing so will void pads warranty. No solution should be left to dry on the pad surface. After the allotted time has passed to ensure disinfection, the pad surface should be wiped again with a wet cloth to remove the remaining chemical or solution.

DOs & DON'Ts

- DO clean all stains promptly.
- DO use only fabric-safe cleaning agents or disinfectants.
- DO use appropriate quaternary or phenolic-type disinfectants if fabric needs to be sanitized.
- DO dilute all disinfectants and germicides in accordance with manufacturer's instructions.
- DO wipe fabric clean with neutral soap suds and lukewarm water.
- DO rinse thoroughly.
- DO towel dry.
- DO allow adequate drying time before returning to service.
- DO use a soft sponge with liquid cleaner as specified on the cleaner manufacturer's product label for hard-to-clean areas.
- DO clean daily to control or prevent odors on long-term incontinent applications.
- DO disinfect blood contamination with a 1:10 dilution of household bleach (5.25% sodium hypochlorite) as recommended by the CDC (Center for Disease Control, US Dept. of Health and Human Service, February 1989); a weaker dilution, e.g., 1:100 may be used, but may not be in accordance with CDC recommendations.
- DO be aware that staining chemicals and cleaning agents can affect fabric strength, finish and color.
- DO inspect pads frequently to ensure that new stains can be treated promptly.
- DO NOT machine wash or machine dry.
- DO NOT use harsh cleaners or solvents.
- DO NOT use cleaning agents designed for use on hard, non-porous or metal surfaces.
- DO NOT use iodophor-type disinfectants, such as Betadine, as they will stain the fabric.
- DO NOT use household bleach or concentrated cleaning agents, disinfectants or germicides without diluting the product according to manufacturer's instructions.
- DO NOT allow any cleaning agent to remain in contact with the fabric or "dwell" on the fabric for a prolonged period of time.
- DO NOT fail to adequately rinse the cleaning agent or disinfectant from the fabric.
- DO NOT fail to towel dry pads after cleaning/rinsing.



Warranty of Gel Pads

- Bryton's surgical table pads are manufacturer warranted against defects in materials for 1 year.
- Alterations, knife cuts, and abuse void manufacturer warranty.
- Warranty does not include damage caused from excessive stretching, twisting or heat.
- Use of surgical pads for other than intended purpose voids warranty.